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1. Press the lenses with the rubber insert from back to front out of the goggles.
2. Press the lenses out of the rubber inserts.
3. Grind the lenses using the plan lens as your pattern with a flat bevel. The size of your lenses must correspond to the size of the plan lenses. The recommended maximum thickness at the edge of your lens is 4 mm.
4. If your lenses are thicker than 4 mm, it is necessary to grind with an automatic bevel. The bevel should be placed towards the back of the lens edge. The bevel curve should be brought as far towards 0 as possible. Finish using your hand edger while taking enough of the lens material off so that the lenses can slip into each groove correctly. We do not recommend using lenses with edges thicker than 6 mm.
5. If the lenses are not watertight, we recommend gluing the lenses in with our silicone glue, article no. 316300.
6. Place the lenses into the grooves of the rubber inserts. Then press the lenses together with the rubber insert into the goggles from the front. It is important that the rubber seal on the front of the goggles is properly inserted. You must be able to see the complete edge of the rubber seal. The rubber seal may not get stuck on the facet of the lenses. If necessary, pull out the edge of the seal.

Attention: The goggle PD changes when the nose width is adjusted. Please determine nose width when purchasing the lenses.

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